

# MEMBRASIN®

MEMBRASIN VAGINAL VITALITY CREAM

Clinically Proven to  
Relieve Vaginal Dryness



**arΩmtech**  
SCIENCE OF NATURAL VITALITY

MEMBRASIN.FI

## PRODUCT INFORMATION

Sea Buckthorn Oil cream 30 ml + applicator

Sea Buckthorn Oil 3% Lactic Acid 1% Sodium Hyaluronate 0,1%

Intimate cream to treat vaginal and vulvar dryness.


Emulsion cream with triple effect:

- 1 Sea buckthorn oil strengthens and protects the mucosa and skin of the intimate area
- 2 Hyaluronate for optimal hydration
- 3 Lactic Acid maintains the normal pH balance

- Hormone and fragrance free cream that nurtures the vaginal mucosa and intimate areas, and alleviates symptoms due to dryness.
- The pH of the cream corresponds to the pH value of healthy vaginal mucosa.
- Medical device product developed and manufactured in Finland
- Efficacy of the cream is clinically proven in a controlled randomized study

### Administration to the vagina (in the evening):

1. Screw the applicator onto the tube, press the cream into the applicator up to the black line (approximately 2.4 grams) and unscrew the applicator from the tube.
2. Insert the applicator into the vagina and press the cream into the vagina. You may also use clean fingers to apply the cream. During pregnancy, apply the cream without using the applicator.
3. Wash the applicator carefully with warm water after use.

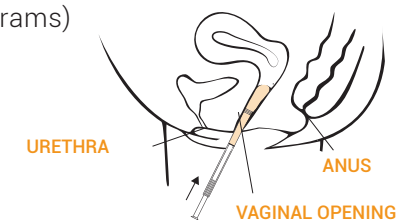

### Application to the external genitalia:

Wash your hands and squeeze a small amount of cream out of the tube, dose the size of your fingertip (approx. 0,2-0,4 g). Apply to the mucous membranes and skin of the genitalia in the evening and after washing.

### Ingredients:

Hippophae Rhamnoides (Sea Buckthorn) Oil (3 %), Lactic Acid (1 %), Sodium Hyaluronate (0,1 %), Aqua, Cetearyl Alcohol, Dicaprylyl Ether, Sodium Lactate, Polysorbate 60, Cetyl Palmitate, Sorbitan Stearate, Helianthus Annuus (Sunflower) Seed Oil, Rosmarinus Officinalis (Rosemary) Leaf Extract, Sodium Dehydroacetate, Sodium Benzoate, Phenoxyethanol

Storage: +5 – +25 °C. Shelf life 6 months after opening  
Carefully read the instructions for use included in the package.

CE 0537